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Flamigel®

This medication

- This medication is a product that affects your health, and its consumption contrary to the instructions poses a risk to you.
- Strictly follow the doctor's prescription, the specified method of use, and the instructions of the pharmacist who dispensed it to you.
- The doctor and the pharmacist are the experts in the medicine, its benefits and risks.
- Do not interrupt the specified period of treatment on your own.
- Do not repeat the prescription without consulting your doctor.
- Keep all medications out of the reach of children.

The Council of Arab Health Ministers
and The Federation of Arab Pharmacists

Indications

Flamigel® is a hydro-active colloid gel for the treatment of minor wounds which supports the fast healing of skin by covering the wound and creating an optimal moist healing environment. **Flamigel®** is used post lasertherapy, on superficial burns (including those caused by radiotherapy) and on minor wounds (e.g. grazes, cuts). **Flamigel®** may also be used in superficial open wounds.

Properties

Flamigel® hydro-active colloid formulation can quickly help soothe the pain of minor wounds due to its cooling effect. **Flamigel®** helps to create optimal healing conditions to accelerate cell renewal, allowing the wound to heal fast and so reduce the likelihood of scarring. **Flamigel®** also cushions the wound to provide a barrier against external contamination.

Flamigel® can be used for dry or wet wounds:

- In case of a dry wound **Flamigel®** turns the dry wound into a moist one ("hydrogel" effect). The dry wound absorbs the water contained in **Flamigel®** and promotes the dissolution of necrotic/damaged tissue.
- In case of a wet wound, the hydrocolloid of **Flamigel®** is activated and consequently **Flamigel®** absorbs excessive fluid from the wound and restores and maintains the water balance in the wound ("hydrocolloid" effect).

How to use

Wash hands before and after treating the wound. Clean and dry the wound gently before applying the gel.

For superficial burns and minor wounds (e.g. grazes, cuts)

Carefully apply **Flamigel®** to the wound (without rubbing) once or twice per day (in case of sunburn apply twice to six times daily).

If necessary, cover the wound with a plaster or dressing and change daily.


For superficial open wounds:

Apply a thick layer of **Flamigel®** (5 mm) to the wound minimising contact with the wound edges.

Cover **Flamigel®** with a sterile non-adhesive dressing and fix with gauze or a bandage. Leave **Flamigel®** in place as long as the gel structure is intact.

When new skin cells start to form and the wound is closing, only apply a thin layer of **Flamigel®** to prevent weakening. If necessary, protect the wound edges.

- Open wounds: change daily.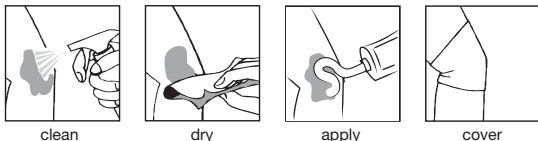

Removing the gel:

- Wash hands before and after removing **Flamigel®**.
- **Flamigel®** can be removed simply and painlessly with wet gauze.

Precautions

- For external use.
- Side-effects such as irritations or allergic reactions are rare.
- Do not use **Flamigel®** when there is a known allergic reaction to parabens or any of the ingredients.
- Do not use **Flamigel®** on the eyelids or in the eyes. If **Flamigel®** comes into contact with the eyes rinse the eyes with running water and consult a doctor.
- On extensive weeping or infected wounds, consult a doctor before using **Flamigel®** because other complementary treatments might be necessary.
- If the wound becomes swollen, reddened or painful, or doesn't start to heal within a few days seek medical advice.

Storage conditions

- Store at room temperature (< 25°C), in a dry place and in the original packaging.
- Close the tube after use.
- Do not use **Flamigel®** after the expiry date shown on the pack. The expiry date (USE BY: Year/Month) is the last day of the indicated month and year.

Ingredients

Acid colloidal hydrocolloid, Arginine, Purified water, Macrogol, Branch Chained Fatty Acid (BCFA), Methyl-p-hydroxybenzoate (E218), Propyl-p-hydroxybenzoate (E216), Disodium EDTA.

Reporting of side effects

- Saudi Arabia:

The National Pharmacovigilance and Drug Safety Centre (NPC)
Fax: +966-11-205-7662
Call NPC at +966-11-2038222, Exts: 2317-2356-2353-2354-2334-2340.
Toll free phone: 8002490000
E-mail: npc.drug@sFDA.gov.sa
Website: www.sFDA.gov.sa/npc

- Other GCC States:

Please contact the relevant competent authority.

How supplied

Tubes of 10g, 15g, 25g, 50g, 100g and 250g.
Not all pack sizes may be marketed.

Legal manufacturer

Flen Pharma NV/SA, Blauwesteenstraat 87, B-2550 Kontich, Belgium
Manufactured by Conforma NV, Zenderstraat 10, B-9070 Destelbergen, Belgium for Flen Pharma NV - MADE IN BELGIUM

Medical Device

Date of issue: 2016/06