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## Wound Gel Dressing

### Indications

Flamigel® wound gel dressing is a hydrogel used for wounds which covers and protects the wound, and creating a moist healing environment for the wound.

### Properties

Flamigel® wound gel dressing can be used for dry or wet wounds.

- In case of a dry wound, Flamigel® turns the dry wound into a moist one. The dry wound absorbs the water contained in Flamigel®, which promotes the dissolution of necrotic/damaged tissue.
- In case of a wet wound, Flamigel® wound gel dressing absorbs excessive fluid from the wound and restores and maintains the water balance in the wound.

### How to use

#### Use of Flamigel®:

Wash hands before and after treating the wound. Clean and dry the wound carefully before applying Flamigel®.

- Dry wounds:  
Carefully apply a thin layer of Flamigel® wound gel dressing (Flamigel®) to the wound (without rubbing) between twice and six times a day.
- Wet open wounds:

Apply a thick layer of Flamigel® (5 mm) onto the wound, avoiding contact with the wound's edges. Cover with a sterile non-adhesive dressing (e.g. Vaseline gauze), and fix with sterile gauze or a bandage. Change daily.

#### Removing Flamigel®:

- Wash hands before and after removing Flamigel®.
- Flamigel® can be removed simply and with less pain using wet gauze, thus avoiding secondary injury to the wound.

### Warning

Do not use Flamigel® for more than 28 days.

### Precautions

- For external use.
- Side-effects such as irritations or allergic reactions are rare.
- Do not use if allergic to any of the ingredients.
- Do not use on the eyelids or in the eyes. If the gel comes into contact with the eyes, rinse the eyes with clean water and consult a doctor.
- Flamigel® is suitable for minor wounds. On extensive weeping or infected wounds, consult a doctor before use because other complementary treatments might be necessary.
- If the wound becomes swollen, reddened or painful, or doesn't start to heal within a few days, seek medical advice.

### Storage conditions

- Store at room temperature (< 25 C), in a dry place and in the original packaging.
- Please close the tube after use.
- Please do not use the product after the expiry date shown on the pack.

### Ingredients

Acid colloidal hydrocolloid, Arginine, Purified water, Macrogol, Cetearylpolycarbanoate liquid, Cetearylpolycarbanoate solid, Methyl-p-hydroxybenzoate (E218), Propyl-p-hydroxybenzoate (E216), Disodium EDTA.

### How supplied

Tube of 10g, 50g and 250g. Not all pack sizes may be marketed.

### Manufacturer

Flen Pharma N.V., Blauwesteenstraat 87, 2550 Kontich Belgium  
www.flenpharma.com - info@flenpharma.com

Manufacturing address:

Zenderstraat 10, B-9070 Destelbergen, Belgium

Tel: +32 3 825 70 63

Fax: +32 3 226 46 58

After-sales service provider:

Profex Shanghai Co., Ltd.

Registered Address:

Rm 1424-3, Level 14, Tower, 1 Jilong Rd, Shanghai Waigaoqiao Free Trade Zone

Office Address:

22nd Floor, No.699, Oriental Center, Nanjing West Road, Shanghai

Tel.: 86-21-62880088

Fax: 86-21-62882222

Postcode: 200041

Registration Certificate No.: CFDA (I)20133644826

Product Standard: YZB/BEL 5642-2013

Date of issue: 04/2017  
0903-729/F  
1110v2