Flamirins®

Isotone gebufferde wondreiniger

Indicaties

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Isotonic buffered wound cleanser

Indications

Flamirins® is used to clean wounds and to loosen adhered dressings. Clean wounds heal better.

Flamirins® mechanically removes foreign bodies and dead tissue from the wound, thus reducing the risk of infection.

Properties

Flamirins® is a solution with the following properties:

- buffered (acid pH)
- isotonic

These properties ensure that any small wound can be cleaned simply, quickly and thoroughly.

Once opened, **Flamirins®** can be reused several times until the indicated expiry date.

Instructions for use

- Remove the cap from the bottle.
- Clean the wound by firmly squeezing the bottle and squirting the liquid into the wound.
- Replace the cap on the bottle.
- Dab the wound dry carefully.
- If necessary, cover the wound with a suitable dressing.

Precautions

- For external use only.
- Keep out of the reach of children.
- Do not use in case of irritation or if allergic to one of the ingredients.
- Do not use as an antiseptic solution.
- Do not use on the eyelids or in the eyes. If **Flamirins®** does come into contact with the eyes, rinse for at least 5 minutes with running water.
- Contact a doctor if the wound becomes worse.

Storage

Store at room temperature (< 25°C) in a dry place and in the original container.

Do not use **Flamirins®** after the expiry date shown on the pack. The expiry date (USE BY: Year/Month) is the last day of the indicated month and year.

Close the bottle properly immediately after use.

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